


ExcelGinate[®] AG

Calcium Alginate Dressing-Antimicrobial Silver

Advanced Antimicrobial Wound Care

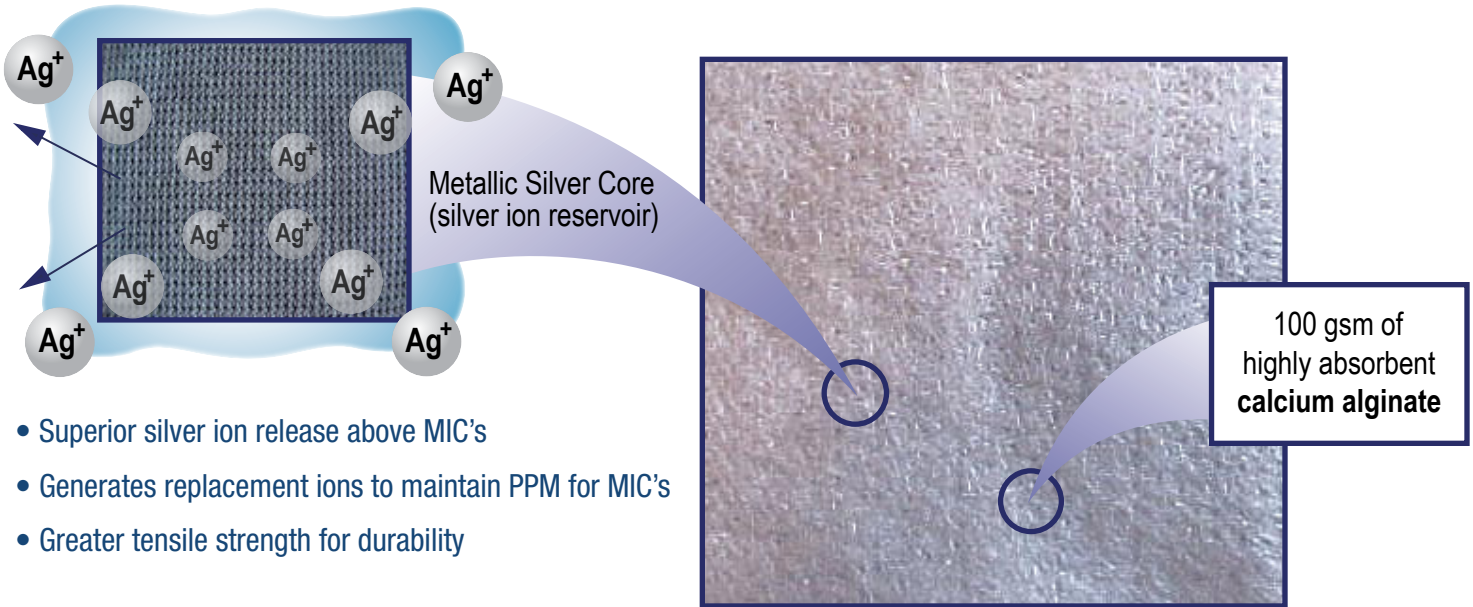

Dressing Construction

- 100 gsm (grams/square meter) of calcium alginate needle punched through a metallic silver plated nylon mesh core.

Double Duty Performance — With a Bonus

- The high gsm provides extra absorbency and exudate management.
- The metallic silver mesh core provides up to 7 days of antimicrobial protection.
- The durable mesh core ensures the dressing high integrity tensile strength, that can be easily cut to fit without falling apart, for assured one piece removal.

Product #	Size	Packaging
MP00830	2 x 2	10/Box, 5 Boxes/Case
MP00832	4 x 4.75	10/Box, 5 Boxes/Case
MP00834	12" Rope	5/Boxes, 5 Boxes/case

MPM
MEDICAL, INC.
The Pain Management Company

Calcium Alginate Dressing-Antimicrobial Silver

MPM ExcelGinate®AG Release Rate (PPM) Comparison

PRODUCT	4hr	8hr	16hr	24hr	48hr	72hr	%
MPM ExcelGinate®AG	11.3	16.9	20.6	25.0	32.2	34.9	--
Aquacel®Ag	15.4	16.7	17.0	18.0	18.7	18.6	- 87.6 %
Silvercel™	11.6	11.8	11.8	11.8	11.9	12.0	- 190.8 %
SilvaSorb™	12.0	13.9	14.0	16.5	17.3	19.1	- 82.7 %

Fiber Strength Comparison (Standard Tensile Burst Test)

Rope	Dry	Saturated	% Decrease	Tensile Strength compared to MPM ExcelGinate®AG
MPM ExcelGinate®AG	60	46	30 %	--
Aquacel®Ag	65	2	3,150 %	- 2,200 %
Silvercel™	35	20	75 %	- 130 %

INSTRUCTIONS FOR USE

Site Preparation

- Debride when necessary and irrigate the wound site in accordance with standard protocols.
- Remove excess solution from surrounding skin.

Dressing Selection


- Select a size of MPM ExcelGinate®AG Dressing that is slightly larger than the wound.

Dressing Application

- 1) Cut using clean scissors or fold the dressing to fit the wound, either side down.
- 2) Loosely pack deep wounds, ensuring the dressing does not overlap the wound margins.
- 3) For heavily exudating wounds, apply to the wound bed directly. For wounds with minimal exudates, apply to a sterile water or saline moistened wound bed.
- 4) Cover and secure MPM ExcelGinate®AG Dressing with a non-occlusive secondary dressing.



Metallic Silver Mesh Core

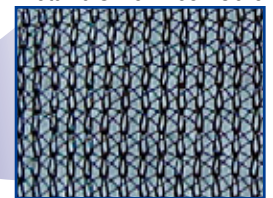

Dressing Change and Renewal

- 1) Dressing change frequency will depend on patient condition and the level of exudates. Initially it may be necessary to change the dressing every 24 hours.
- 2) Reapply MPM ExcelGinate®AG Dressing when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.
- 3) Gently remove the secondary dressing.
- 4) If the wound appears dry, saturate the dressing with sterile water or normal saline solutions prior to removal.
- 5) Gently remove the dressing from the wound bed and discard.
- 6) Irrigate the wound site with a suitable wound cleanser prior to application of a new dressing.

www.mpmmed.com
To Order: (800) 232-5512